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

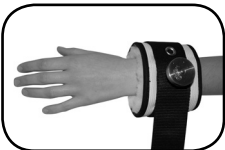
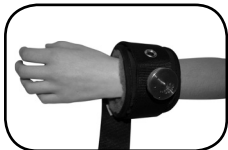
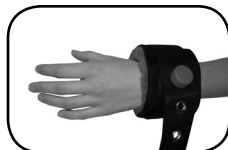
TOBILLERAS

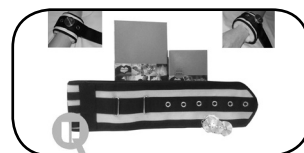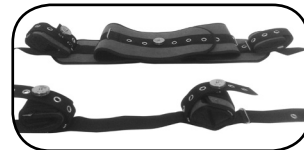KIT COMPLETO

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ATTENTION: QUALIFIED STAFF MUST ALWAYS SUPERVISE THE INSTALLATION.
Incorrect use of any product may cause severe lesions or death.

PRODUCT SELECTION:

Take a look at our Ubiotex catalog to help you choose the correct product that fit the individual needs of patients.

PREVIOUS RECOMMENDATIONS FOR USE:

- A complete clinical examination of the patient (symptoms and needs) must be carried out.
- The consent of the patient or legal guardian is required before using the product. The use of sling products must be explained to the patient or guardian in question to ensure their cooperation.
- Sling products must be used according to the Individualized Care Plan (ICP) of the patient in question. ICP constitutes an evaluation carried out by an interdisciplinary team which may include (without any restriction): Occupational or physical therapy OT, Medicine and Social Services. ICP Plan should include recommendations about restorative nursing, patient's release and pain's prevention.

NOTE: Considering that patient's behavior is not 100% predictable, no product is 100% infallible. Patient's safety requires a regular re-evaluation and supervision. A product that has worked in the past may be inappropriate if mental and physic conditions of the patient have been modified. NEVER use an unsafe product. Please contact the responsible medical authority if you have questions about patient's safety.

ADDITIONAL WARNINGS:

1. A regular supervision of immobilization process of the patient must **ALWAYS** be carried out by qualified staff. A constant supervision should be required to:
 - Patients with an aggressive and agitated behaviour.
 - Patients who are in danger of swallowing its own vomit. This fact includes patients in a supine position or being unable to seat by themselves.
 - Be ready to react when a danger sign occurs. These patients require both a regular examination and an evaluation of their physical and psychological status.
2. This product must **NEVER** be modified nor repaired. The condition of the product must **ALWAYS** be verified before using it (emergence of flaws or broken pieces, orifices, cuts, deteriorations, dysfunctions with the magnet or Iron Clip closures, etc.). Do **NOT** use dirty or damaged products. Therefore, this fact may cause severe lesions or even death may occur. Damaged products must be disposed of.
3. Straps must **ALWAYS** be fixed to an immobile part of the bed or the chair's edge out of reach of patients (by making quick loops or using buckles). These kinds of slings provide better release process in case of accident or fire risk. Straps mustn't press or slip. If this occurs, the patient can slip from the chair or bed (emergence of severe lesions or suffocation). Removing slings from patients constitute an important activity in case of fire risk or catastrophes in facilities. Straps can be cut with scissors in case of emergency.
4. Ubiotex products must **NEVER** be used on toilets, chairs or furniture not allowing its correct use as indicated in the Directions for use.
5. Ubiotex products must **NEVER** be exposed to flames, fire, smoking materials or high heat sources. Some products can melt or get burned. The smoking/ no smoking policy must be strictly implemented.
6. **NONE** Ubiotex product can be used as a security belt in a vehicle in motion. The design of Ubiotex products doesn't support collisions or sudden stops.

⚠️**WARNING!!** Try zippers or closures (buckles, magnet or iron clip) before each use. Dispose of the product if it doesn't work correctly.

STORAGE AND MANIPULATION:

- This product has been designed to be used in regular interior areas, hospitals, etc.
- This device can be stored at ambient room temperatures with normal humidity levels.
- Avoid humidity excess or humidity conditions that may damage the product's materials.

BED RESTRAINTS

EN

DESCRIPTION

Mechanical or physic restraints are an exceptional therapeutic measure to partially or generally immobilize most of patient's body in question. In this way, the product provides safety to the patient and to the rest of the people who surrounds him. This product infringes the autonomy right of patients (the patient's freedom is restricted) which entails a fundamental protection and constitutional right restriction of people suffering from body operations. In this way, this product is only recommended due to the failure of other alternatives.

INDICATIONS

- Imminent damage prevention of the patient if other measures (verbal or pharmacological contention) have failed.
- Prevention of surrounding damages (family, patients, health professionals, facilities...).
- The prescribed treatment is completely ensured when this one is associated with aggressive behaviours.
- Contingency in the behavioural therapy of dangerous behaviours.
- Geriatric and psychic disabled patients.
- Others:
 - Minimal reduction of treatment during the observation of delirium and psychotic patients.
 - Psychotic attacks in pregnant women.
 - Damage prevention during urgent treatments.
 - Aggression and self-mutilation prevention in case of mental disabled patients.

Faced with this serious situation, our company Ubio Quality life has developed a huge range of products and accessories to achieve a greater efficacy during the immobilization processes of patients at minimal risk.

CLASSIFICATION

UBIOTEX BED RESTRAINTS are considered a type I medical device (rule 1). The conformity evaluation was carried out by following the indications in Annex VII (Declaration CE of Conformity) of the Directive 93/42/CEE and the Directive 2007/47. In accordance with Annex IX of the Directive 93/42/CEE and its subsequent modifications, the health product is defined as:

- Non-Invasive Health Product

Rule 1. – All non-invasive products are included in the Type I category unless one of the following rules is applicable.

MODELS

1.- ABDOMINAL OR PERINEAL BED RESTRAINTS

Three qualities are available depending on the use and washing of the fastening systems. The function of all of them is to prevent the patient from getting out of bed, but allowing him a good mobility so that he does not feel immobility: it allows to raise the trunk, and to move freely the leg, as well as to turn in the bed to right and left , But without protruding from it or falling.

- 1.- Polypropylene. Composition: 100% polypropylene, metallic hooks, closure system (magnet, ironclip or buckle). Polypropylene bias.
- 2.- Padded. Composition: imitation suede, velour and foam. Reinforced polypropylene fabric, metallic hooks, closure system (magnet, ironclip or buckle). Polypropylene bias.
- 3.- Canvas. Composition: 100% cotton. Reinforced polypropylene fabric, metallic hooks, closure system (magnet, iron clip or buckle). Polypropylene bias.

BED RESTRAINTS

EN

Two belt models are available: abdominal and perineal. The difference between both of them is that the perineal model prevents the patient from sliding downwards (more fastening to the patient). In the same way, our company disposes of a model (BASIC) which permits the lateral mobilization of the patient. The product is composed of an abdominal strap that keeps the patient lying on his back. The above mentioned models dispose of different closure systems: mechanical system, iron clip system and buckle system. The bed restraint belts are available in different measures: 90, 105 y 135 cm.

Our company can manufacture any other size upon customer request.

Three sizes are available (waist size): S (60-90cm), M (75-125cm), L (105-145cm).

2.- WRISTBANDS FOR BED

This product is sold in pairs. In this way, these products can adapt themselves to bed or wheelchairs. The main function is to restrict the upper extremities mobility (relax and safety). All models dispose of an interior damping to avoid lesions and also to provide a greater comfort.

Our company also disposes of four models available in different materials and closure systems depending on the needs of the patient and health professionals:

- 1.- Suapel: in three colors, White, Navy, Gray. Tape closure system. Composition: terlenka fabric and SANITIZED polyester (anti-dust, anti-bacterial, anti-fungal, anti-bedbug, soft to the touch, breathable and washable).
- 2.- Polypropylene: Closing system: Magnet, IronClip.
- 3.- Padded: Greater security and comfort, being doubly cushioned. Closing system: Magnet, IronClip.
- 4.- Canvas: Closing system: Magnet, IronClip.
- 5.- Sujetax: Safety and comfort. Closing system: Buckles.

3.- HEEL-PADS FOR BED

This product is sold in pairs. The main function is to restrict the upper extremities mobility (relax and safety). All models dispose of an interior damping to avoid lesions and also to provide a greater comfort. Our company also disposes of three models available in different materials and closure systems depending on the needs of the patient and health professionals:

- 1.- Polypropylene: Closure system: Magnet, Iron Clip.
- 2.- Padded: Greater safety and comfort thanks to its double padded system. Closure system: Magnet, Iron Clip.
- 3.- Canvas. Closure system: Magnet, Iron Clip.

4.- BED RESTRAINTS COMPLETE PACK

This pack offers greater fastening and security to health professionals. This product also prevents the self-mutilation and escape of patients (extreme situations). In contrast, the product notably reduces the mobility (partially or total) thereof depending on the chosen pack. In this way, the product permits a 90° movement of the trunk from right to left.

Two models are available:

- 1.- Restraint belt with wristbands and fixed heel-pads, (this model offers greater mobility reduction)
- 2.- Belt, wrist straps and independent anklets. And in both models we have the following materials:
 - Polypropylene
 - Padded material
 - Canvas

Please contact us for any other measure.

BED RESTRAINTS

EN

5.- ABDOMINAL BED THORAX RESTRAINT BELT

Padded bed thorax restraint belt (single bed 90cm) S/M

6.- ACCESSORIES

IRON CLIP closure: stainless steel mechanical system.

This product is specially indicated to be used with restraint belts.

The product can be opened with the special key but also with a 50 cent or a 1 Euro coin. In this way, this confers a greater safety to free the patient during an emergency.

Stainless steel surgical material.

The components are: one-block pivot with grip.

The hook is made of one-piece stainless steel.

- Reference: 409400 - Button
- Reference: 409405 - Key
- Reference: 409350 - Pack (3 buttons + key)

MAGNET closure: magnetic plastic system

This is the classic model composed of a button, a metallic rod and a magnetic key.

The main advantage is that the securing button can be quickly placed and remains immediately closed.

The product can only be opened with the magnetic key. In this way, the unwanted opening (unauthorized manipulations) of the restraints from patients or unauthorized personal is avoided.

- Reference: 409100 - Button
- Reference: 409200 - Key
- Reference: 409300 - Pack (3 buttons + key)

Buckle (5 cm)

Trident buckle composed of two pieces (male and female).

The product has been designed to be used with restraint belts (high impact-resistant plastic).

100% ABS plastic product.

- Reference: 0501

Securing buckle (5 cm)

The same as the trident buckle, this product is composed of two pieces (male and female). This buckle has been designed to be used in restraint belts (high impact-resistant plastic). This product also disposes of a three-number combination closure system to close the buckle in a safe way. The opening code is 0000.

This product is indicated to be used with restraint belts which require a safety system to avoid unwanted openings (from patients or unauthorized personal).

100% ABS plastic product.

- Reference: 0508

BED RESTRAINTS

EN

SIZES

Different sizes are available depending both on the bed measures and the size of the patient. Our company offers the manufacture of special sizes upon customer request.

INSTALLATION

The following steps must be considered to install the Bed Restraints Pack:

- Place the belt on the bed. Verify the correct adaptation of the product to the measures of the bed.
- Use lateral straps to fix the product to the bed with the closure system of the kit (buckles). The fastening must be stable (without sliding upwards or downwards).
- Install the wristbands and heel-pads by following the same steps of the restraint belt.
- The mattress must not be pressed nor deformed.
- Make the bed with the bed protectors.
- Place the patient on the belt and fix him with the closure system of the product.
- The fastening must be carried out in a firm way but without making pressure on the patient.
- Place the wristbands on the hands and then the heel-pads on the heels.

In any case, our company recommends that the use and adjustment of the product must be carried out by qualified personnel: Orthopaedic technicians or health professionals.

CLEANING/DISINFECTION

PADDED MATERIAL: Hand and machine washable at 40°C. Do not use abrasive products. Dry in a horizontal position. Do not expose to direct heat sources as heaters.

CANVAS MATERIAL: Hand and machine washable at 95°C. Do not use abrasive products. Do not iron.

POLYPROPYLENE MATERIAL: Hand and machine washable at 95°C. Do not use abrasive products. Do not iron.

SANITIZED MATERIAL: Hand washable with cold water. Do not use abrasive products. Use a soft program in washing machine. Dry in a horizontal position. Do not iron. Do not use bleach. Do not dry clean.

SECURITY

The product must fit the necessities of the user and also the dimensions of the chair/armchair where is going to be used.

ATTENTION: The professional supervision by an expert is recommended.

BED RESTRAINTS

EN

PRECAUTIONS

Do not use caustic, abrasive, solvent or sharp objects in direct contact with the products.

Test regularly the product's deterioration (orifices, cracks, surface's modification, etc.) to verify the correct function of its technical properties.

Avoid prolonged exposure to intense heat sources or flames.

MATERIAL'S PRECAUTIONS

Avoid prolonged exposure to liquids and humidity conditions with the product.

MAINTENANCE

Store the products preferably in a flat position and protected against sunlight and humidity.

RECYCLING

Do not burn nor throw away in the nature outside the correct places for this action.

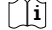
GUARANTEE

The guarantee period of the products is two years from the date of the original purchase. This guarantee does not cover the regular deterioration of the product and does not substitute legal guarantees. Please contact the Quality Department of the company to any incident that may occur to correctly resolve it.

LIFETIME OF THE PRODUCT

The regular lifetime of the products is about a year.

LEGEND AND LABEL SYMBOLS

| | |
|---|--|
|  | Maker. |
|  | Serial number. |
|  | Catalogue number. |
|  | Read the instructions for use first. |
|  | Certificate of conformity with Directive 93/42 / EEC |
|  | Caution. Read the instructions for use first. |



| | | MODEL | CLOSURE | MEASURE | REFERENCE | INSTRUCTIONS |
|------------------|------------------|-----------------------|----------------------|---|--|---|
| BED BELTS | ABDOMINAL | POLYPROPYLENE "BASIC" | BUCKLE | 90 | 401115 | 1. Place the abdominal strap on the waist of the use when the patient is placed on the bed. 2. Place the polypropylene straps under the bed and close them with the buckle. |
| | | PADDED "BASIC" | BUCKLE | 90 M-L | 401230-401232 | |
| | | POLYPROPYLENE | BUCKLE | 90 M-L | 401100-401102 | 1. Place the belt on the bed. Place the patient and close the belt over the waist zone, adjust it without putting pressure and close it with the closure system chosen: buckle, magnet or iron clip. 2. The bed restraint belt disposes of a three-button pack and a key (only with magnet or iron clip closure systems). In buckle closure systems, the buckles are integrated in the belt. One of the buttons closes the abdominal strap on the patient's waist. 3. The polypropylene straps must be placed on the mattress base. Close them with the closure system chosen: magnet or iron clip. |
| | | | | 105 M | 401111 | |
| | | | 135 M-L | 401110-401112 | | |
| | | MAGNET | 90 S-M-L | 401121-401120-401122 | | |
| | 105 M-L | 401105-401107 | | | | |
| | 135 M-L | 401127-401129 | | | | |
| | IRONCLIP | 90 M-L | 401150-401152 | | | |
| | 105 M-L | 401130-401132 | | | | |
| | 135 M-L | 401154-401156 | | | | |
| | PADDED | BUCKLE | 90 S-M-L | 401201-401200-401202 | | |
| 105 M-L | | | 401205-401207 | | | |
| 135 M-L | | 401220-401222 | | | | |
| MAGNET | 90 S-M-L | 401240-401242-401244 | | | | |
| 105 M-L | 401245-401247 | | | | | |
| 135 M-L | 401250-401252 | | | | | |
| CANVAS | MAGNET | 90 S-M-L | 401261-401260-401262 | | | |
| | 105 M-L | 401270-401272 | | | | |
| 135 M-L | 401280-401282 | | | | | |
| IRONCLIP | 90 M-L | 401400-401402 | | | | |
| | 105 M-L | 401410-401412 | | | | |
| 135 M-L | 401420-401422 | | | | | |
| PERINEAL | PADDED | MAGNET | 90 M | 402510 | 1. Follow the previous instructions. 2. Place the perineal strap between the legs and close it with the abdominal strap (with the same button). | |
| | | IRONCLIP | 90 M | 402500 | | |
| | CANVAS | MAGNET | 90 M | 402520 | | |
| POLYPROPYLENE | MAGNET | 90 M | 401180 | | | |
| | | 90 M | 401175 | | | |
| IRONCLIP | MAGNET | 90 M | 402550 | | | |
| | | 90 M | 402550 | | | |
| PADDED CHEST | MAGNET | 90 M | 402550 | Follow the steps of the abdominal belt closing magnet / ironclip. The difference is that it has straps that close next to the abdominal band. | | |

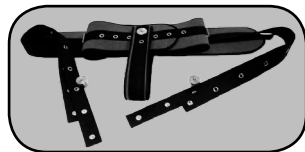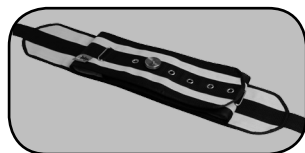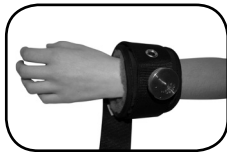
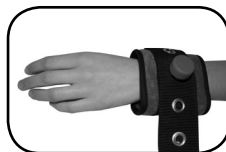
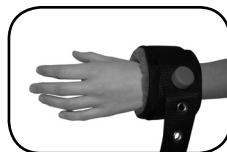
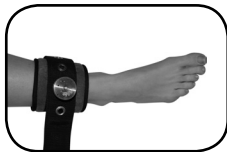
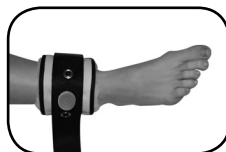
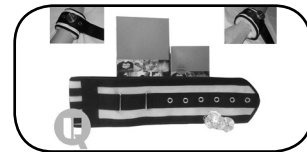
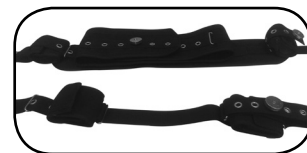
| MODEL | | CLOSURE | REF. | INSTRUCTIONS |
|------------------|--|--|--|---|
| WRISTBANDS | POLYPROPYLENE FIXED WRISTBANDS | IRONCLIP | 401164 | <ol style="list-style-type: none"> 1. Place the belt on the bed. Once the belt is positioned, position the user, close the belt around the user's waist area, adjust tightly and close. 2. Comes with a set of 3 buttons and 1 key of which 1 closes the abdominal band on the waist of the patient. The wristbands are incorporated into the belt with 2 more buttons to immobilize the wrists. 3. Once the abdominal band has been adjusted, take the polypropylene strips to the mattress base and close. |
| | SUAPEL (PAIR) | WHITE STRAPS NAVY STRAPS GREY STRAPS | 111100 111200 111202 | <ol style="list-style-type: none"> 1. Place the suapel (velvety material) strap around the wrist and adjust it firmly with the Velcro closure system. 2. Take the longest strap and place it around the wrist. Place the strap inside the 3. Attach the remaining strap to the wheelchair. |
| | PADDED (PAIR) | IRONCLIP | 303305 303114 | |
| | POLYPROPYLENE (PAIR) | IRONCLIP MAGNET | 303102 303100 | <ol style="list-style-type: none"> 1. Place the strap around the wrist. 2. Adjust it by matching the hooks to close it with the magnet or iron clip closure system. |
| | CANVAS (PAIR) | IRONCLIP MAGNET | 303108 303110 | |
| | SUJETAX (PAIR) | BUCKLES | 111210 | <ol style="list-style-type: none"> 1. Place the band around the wrist. Fit and glue the strap thus securing the wrist strap. 2. Pick up the polypropylene tape and tie and adjust to the desired height. |
| HEEL-PADS | POLYPROPYLENE (PAIR) | BUCKLES MAGNET IRONCLIP | 404100 404300 404350 | <ol style="list-style-type: none"> 1. Place the strap around the heel and adjust it by matching the hooks to close it with the magnet or iron clip closure system. 2. In buckle systems, adjust the strap around the heel with the Velcro closure and fix it to the mattress base (buckle system). 3. Adjust the straps and fix them to the mattress base. 4. Repeat this step with both heels. |
| | PADDED (PAIR) | BUCKLES MAGNET IRONCLIP | 404200 404500 404550 | |
| | CANVAS (PAIR) | MAGNET IRONCLIP | 404400 404450 | |
| | PADDED FIXED (UNITY) | IRONCLIP | 404600 | <ol style="list-style-type: none"> 1. Place the band around the ankle and adjust by matching the ears to close the ironclip. 2. Tighten the bands and close the bed frame. |
| COMPLETE PACK | POLYPROPYLENE | IRONCLIP T/M IRONCLIP T/L | 401190 401192 | Follow the above steps. |
| | PADDED | IRONCLIP T/M | 401299 | |
| | CANVAS | IRONCLIP T/M IRONCLIP T/L | 401397 401399 | |
| ABDUCTOR CUSHION | POLYPROPYLENE STRAPS WITH VELCRO | 705150 | <ol style="list-style-type: none"> 1. Place the abductor cushion between the legs. 2. Pass the straps in polypropylene under the legs of the patient. 3. Adjust the product with the Velcro and fix it to the abductor cushion. | |

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